
Filter/Ventil Set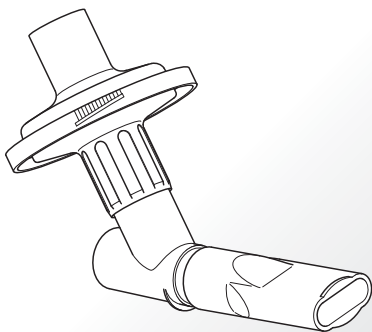

CE 0123

de	Gebrauchsanweisung.....	3
en	Instructions for use	23
da	Brugsanvisning.....	42
el	Οδηγίες χρήσεως.....	60
es	Instrucciones de uso	81
fi	Käyttöohje	101
fr	Notice d'utilisation.....	120
it	Istruzioni per l'uso	140
nl	Gebruiksaanwijzing	160
no	Bruksanvisning.....	180
pt	Manual de instruções	198
sv	Bruksanvisning.....	218
tr	Kullanım Talimatları.....	236

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Instructions for use

Information as of: 2012-08. We reserve the right to make technical changes.

IMPORTANT INFORMATION	25
General	25
Information about the instructions for use	25
Structure of safety instructions	25
Hygiene	26
PRODUCT DESCRIPTION	27
Components	27
Material information	27
Product combinations	28
Intended use	28
Description of function	28
Operating life	28
INHALATION	29
Assembling the PARI Filter/Valve Set	29
Preparing for inhalation	30
Performing the inhalation	31
CLEANING AND DISINFECTION AT HOME	32
Preparation	32
Cleaning	32
Disinfection	32
Visual inspection	33
Drying and storage	33
HYGIENIC RE-USE IN THE PROFESSIONAL ENVIRONMENT	34
General	34
Processing limits	35
Preparation	35
Cleaning and disinfection	35
Sterilisation	38
Visual inspection	39
Storage	39

MISCELLANEOUS.....	40
Disposal	40
Contact	40
SPARE PARTS AND ACCESSORIES.....	41

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. Also read the instructions for use of all other products used with this accessory (compressor, nebuliser, etc.) carefully and completely.

If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

1.2 Information about the instructions for use

All instructions for use can be retrieved on the internet at www.pari.de/en/products (on the respective product page), or they are available in hardcopy form on request from PARI GmbH. **Check for updated versions of all instructions for use at regular intervals.**

1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.4 Hygiene


Observe the following hygiene instructions:

- Only use a cleaned, disinfected and dried PARI Filter/Valve Set for the inhalation therapy. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection
- Each PARI Filter/Valve Set must only be used at home by a single patient for hygiene reasons.
- Before every use and cleaning wash your hands thoroughly.
- Make absolutely sure you also carry out cleaning, disinfection and drying before using the device for the first time.
- For cleaning and disinfection always use fresh tap water of drinking water quality.
- Make sure all components are dried properly whenever you have performed cleaning or disinfection.
- Do not keep the PARI Filter/Valve Set and accessories in a damp environment or together with damp objects.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.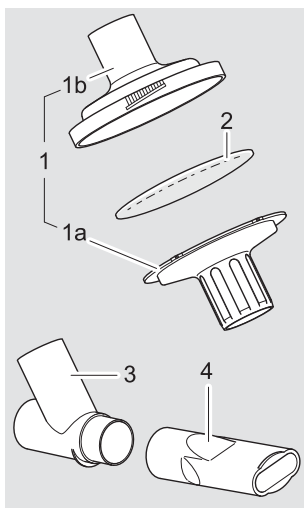

- (1) Exhalation filter
- (1a) Exhalation filter lower part
- (1b) Exhalation filter upper part (with expiratory valve)
- (2) Filter pad¹⁾
- (3) Y-connection
- (4) Mouthpiece without expiratory valve

1) For single use only, not reusable. Not included in the PARI Filter/Valve Set with item no. 041G0501.

2.2 Material information

With the exception of the expiratory valve, which is made from silicone, all parts of the PARI Filter/Valve Set are made from polypropylene.

2.3 Product combinations

The PARI Filter/Valve Set can be used in conjunction with the following nebulisers: PARI LC Family, PARI LL, PARI TIA-S, PARI LC SPRINT, eFlow®*rapid*.

2.4 Intended use

The PARI Filter/Valve Set is an accessory for PARI nebulisers (LL, LC Family and LC SPRINT). The PARI Filter/Valve Set protects the ambient air from contamination with medication (e.g., antibiotics). Together with the inspiratory valve on the nebuliser, the integrated expiratory valve (exhalation filter) controls the flow of medication and significantly reduces the discharge of aerosol into the environment (approx. 95%) during exhalation.

2.5 Description of function

The PARI Filter/Valve Set is constructed (designed) in such a way that the patient's exhaled air must pass through the exhalation filter before exiting the nebuliser. The filter pad located in the exhalation filter removes excess medication aerosol from the exhaled air and prevents this aerosol from being released into the environment.

2.6 Operating life

Replace the PARI Filter/Valve Set after a year at the latest as changes can occur in the material of the PARI Filter/Valve Set over time. This might impair the effectiveness of the treatment. The filter pad must be replaced after every inhalation session.

3 INHALATION

3.1 Assembling the PARI Filter/Valve Set

CAUTION


The filter pad absorbs excess aerosol to prevent it from escaping into the ambient air. After an inhalation session, the filter pad has no more absorption capacity. In order to avoid contaminating the ambient air and thus also creating a **danger to health** for other individuals present, use a new filter pad for each inhalation session.

- Place a filter pad (2) in the exhalation filter upper part (1b).

NOTE

Use only original filter pads [See also: 7 SPARE PARTS AND ACCESSORIES].


- Insert the exhalation filter lower part (1a) in the exhalation filter upper part with the snap locks in the holes provided.
- Close the exhalation filter by turning its lower part clockwise as far as it will go (bayonet closure).
- Insert the Y-connection (3) into the exhalation filter.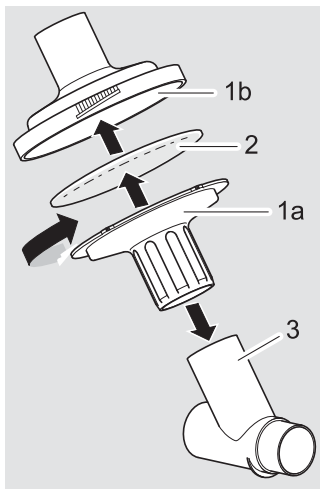

3.2 Preparing for inhalation

- Assemble the nebuliser you are using.
- Fill the nebuliser with the medication.

To do this, follow the instructions for use of the nebuliser you are using.

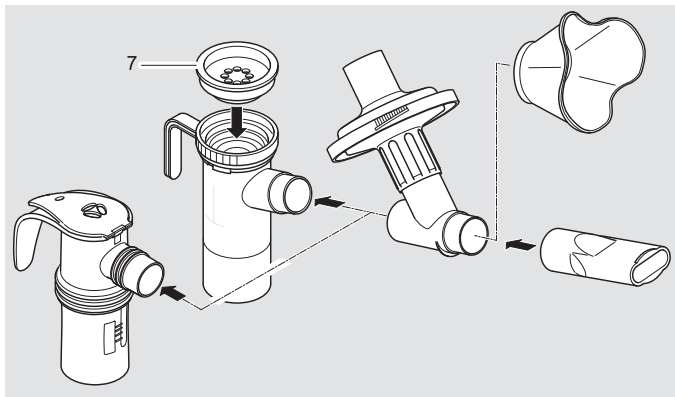- Attach the PARI Filter/Valve Set to the nebuliser.
- Attach the mouthpiece or baby mask to the free end of the PARI Filter/Valve Set.

NOTE

With the PARI Filter/Valve Set, only the mouthpiece that is supplied with the set must be used, and an expiratory valve must not be used; or a mask without a valve system must be used. If a mouthpiece or mask is used with an expiratory valve, the PARI Filter/Valve Set will be completely ineffective.

- Attach the inspiratory valve (7) to the nebuliser if necessary (not necessary if you are using a nebuliser with built-in inspiratory valve, e.g., PARI LC SPRINT).

3.3 Performing the inhalation

- Before the patient starts inhaling, ensure that all parts are firmly connected to each other.
- Perform the inhalation as described in the instructions for use of the nebuliser you are using.

4 CLEANING AND DISINFECTION AT HOME

The PARI Filter/Valve Set must be cleaned after each use and disinfected at least once a day.

4.1 Preparation

Dismantle the PARI Filter/Valve Set into its individual parts:

- Open the exhalation filter (bayonet closure).
- Take the filter pad out and dispose of it.

4.2 Cleaning

- Thoroughly clean all components (without the filter pad) in warm tap water and dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose dirt (the brush must be reserved exclusively for this purpose).
- Rinse all parts thoroughly under warm running water.
- You can remove excess water more quickly by shaking all parts.

4.3 Disinfection

Disinfect the disassembled PARI Filter/Valve Set (without the filter pad) immediately after cleaning (parts can only be disinfected effectively if they have been cleaned first).

CAUTION

A damp environment may encourage the growth of bacteria. Therefore, remove all parts from the pot or disinfector as soon as disinfection has finished. Dry the parts. The **risk of infection** is reduced when the parts are dried completely.

In boiling water

- Place all the individual parts in boiling water for at least 5 minutes.

Use a clean pot and fresh drinking water.

- i** If your water is very hard, scale may be deposited on the plastic parts as a milky coating. This can be prevented by using softened water.

NOTICE

Plastic will melt if it comes into contact with the hot base of the pot. Therefore, make sure there is plenty of water in the pot. This way you will avoid **damaging the components**.

Using a standard thermal disinfectant for baby bottles (not a microwave oven)

Disinfection must last at least 15 minutes. Regarding disinfection and the quantity of water required for this, it is essential to follow the instructions for use of the disinfectant you are using. Always check carefully that it is clean and fully functional.

4.4 Visual inspection

Inspect all product components after each cleaning/disinfection. Replace any broken, misshapen or seriously discoloured parts.

4.5 Drying and storage

- You can remove excess water more quickly by shaking all parts.
- Place all parts on a dry, clean and absorbent surface and let them dry completely.
- When drying, between applications, and in particular during long breaks, wrap the PARI Filter/Valve Set in a clean, lint-free cloth (e.g. a tea-towel) and keep in a dry, dust-free environment.
- Store the PARI Filter/Valve Set safely out of continuous direct sunlight.

5 HYGIENIC RE-USE IN THE PROFESSIONAL ENVIRONMENT

5.1 General

NOTICE

Ensure that the system is processed for re-use only with the application of adequately validated device- and product-specific procedures, and that the validated parameters are maintained during each cycle. The equipment used must be maintained regularly.

Follow the schedule defined below for hygienic re-use:

- Dispose of the used filter pad **after every inhalation session.**
- Clean and disinfect the PARI Filter/Valve Set **after every inhalation session.**
- Clean, disinfect and sterilise the PARI Filter/Valve Set **before every change in patient.**

The PARI Filter/Valve Set must be processed for re-use with the application of suitable, validated methods in such manner that the success of these methods can be monitored reliably and the safety and health of patients is not threatened.

CAUTION

Ensure that the components are allowed to dry sufficiently after each processing step. Condensation or residual wetness can present an increased risk through the growth of bacteria.

The following text describes the methods validated by PARI, with which plastic PARI parts can be processed effectively.

Ensure that the desired results are achieved consistently by the processing methods employed by your staff with the equipment and chemicals used. Take particular care to ensure that the

processing procedure you select is effective if it must differ from our validated procedures (e.g., because the equipment and chemicals described are not available). When selecting the chemicals for use, make sure that they are suitable for cleaning or disinfecting medical products made from the materials indicated [See also: 2.2 Material information]. Bear in mind the manufacturer's dosing information and instructions for safe handling of chemicals.

To assist you in processing the system for hygienic re-use, a checklist may be obtained from PARI GmbH (on the internet at www.pari.de/en/products or from the PARI Service Center [See also: 6.2 Contact]).

5.2 Processing limits

The PARI Filter/Valve Set can be sterilised up to 300 times and used for up to one year.

Do not expose the PARI Filter/Valve Set to temperatures above 137 °C.

5.3 Preparation

Dismantle the PARI Filter/Valve Set into its individual parts:

- Open the exhalation filter (bayonet closure).
- Take the filter pad out and dispose of it.

5.4 Cleaning and disinfection

Clean and disinfect immediately after use. Ideally, a mechanical procedure (instrument dishwasher) is to be used.

NOTICE

When selecting the cleaning agent and/or disinfectant, ensure that it is compatible with the materials used, **otherwise damage to the product components** cannot be ruled out.

Mechanical cleaning and disinfection

EQUIPMENT

- Cleaning and disinfection device in conformance with DIN EN ISO 15883
- Cleaning agent
- Neutralising agent if necessary

PROCEDURE

- Arrange the components so that they can be cleaned most effectively.
- Select a suitable cleaning programme and agent.

VALIDATION

The method has been validated using:

- G7836 CD cleaner-disinfector produced by Miele, programme Vario TD.
- Standard baskets manufactured by Miele
- Neodisher Mediclean forte 0.5% and Neodisher Z for neutralisation (from Dr. Weigert)

DRYING

Even if the cleaner-disinfector is equipped with a drying function, make sure that no residual moisture is left in the components. If necessary, remove any residual moisture by shaking the parts and allow them to dry completely.

Manual cleaning

As an alternative to mechanical cleaning and disinfection, the following manual method can be used:

EQUIPMENT

- Cleaning agent
- Brush
- Warm water

PROCEDURE

- Place all the parts in the cleaning solution.
- Clean the parts thoroughly with a brush.
- Thoroughly rinse the parts under hygienically tested water and leave them to dry completely on a clean, dry and absorbent surface.

VALIDATION

The method has been validated using:

- Korsolex® Endo Cleaner 0.5% (Bode)
- approx. 50 °C warm water
- Duration of cleaning procedure: approx. 5 minutes

Manual disinfection

For manual disinfection, select a suitable instrument disinfectant that has been proven to be effective and compatible with the cleaning agent used.

EQUIPMENT

- Disinfectant
- Water

CAUTION

PARI product components are not suitable for treatment with instrument disinfectants based on **quaternary ammonium compounds**. Quaternary ammonium compounds tend to become concentrated in plastic materials, which can cause **incompatibility reactions** in patients.

PROCEDURE

- Place all the parts in the disinfection solution.
- Allow the solution to take effect as indicated in the instructions for use of the disinfectant.
- Thoroughly rinse the parts under hygienically tested water and leave them to dry completely on a clean, dry and absorbent surface.

VALIDATION

The method has been validated using:

- Korsolex® Basic 4% (Bode)
- Treatment time: 15 minutes

5.5 Sterilisation

NOTICE

A validated sterilisation process can only be carried out on products that have been cleaned and disinfected.

EQUIPMENT

Steam steriliser – preferably with fractionated pre-vacuum – in accordance with DIN EN 285 or DIN EN 13060 (type B)

PROCEDURE

- Pack all of the disassembled parts in a sterile barrier system in accordance with DIN EN 11607 (e.g., foil-paper packaging).

Maximum sterilisation temperature and holding time:

121 °C, at least 20 min. or

132 °C / 134 °C, at least 3 min.

VALIDATION

The method has been validated according to DIN EN ISO 17665-1 using:

- Euro-Selectomat 666 from MMM (fractionated pre-vacuum method)
- Varioklav 400 E from H+P Labortechnik (gravitation method)

5.6 Visual inspection

Inspect all product components after each cleaning/disinfection. Replace any broken, misshapen or seriously discoloured parts.

5.7 Storage

Store the PARI Filter/Valve Set in a dry, dust-free place away from sources of contamination. It is recommended to use sterile packaging.

6 MISCELLANEOUS

6.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

6.2 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)
+49 (0)8151-279 220 (international)

7 SPARE PARTS AND ACCESSORIES

Description	Item no.
PARI Filter/Valve Set	041G0500
Exhalation filter	041B0519
Y-connection	041E0530
Mouthpiece (without expiratory valve)	012E1720
BABY mask, size 0 (for premature babies)	025G0700
BABY mask, size 1 (for age up to 1 year)	041G0701
BABY mask, size 2 (for age 1-3 years)	041G0702
BABY mask, size 3 (for age approx. 4 years and older)	041G0703
Filter pads, pack of 30	041B0522
Filter pads, pack of 100	041B0523
Filter pads, pack of 1000	041B0524

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