EN **ZEN-O lite™**  
**PORTABLE OXYGEN CONCENTRATOR**  
**QUICK USER GUIDE**

Use the device in its carry bag and in an upright position.  
Always use in a well ventilated location.

1. Open the battery compartment by pulling back the release latch, then insert the Zen-O lite™ battery.

2. Close the battery compartment ensuring the cover latches to the device.

3. Connect the AC power supply to your Zen-O lite™ concentrator.

**IMPORTANT: See user manual before use**

The package contains accessories and a user manual to get you started. Check to make sure you have the following: concentrator, battery, carry bag, AC and DC power cords. Always inspect the device and its accessories for any sign of damage before use.

4. Plug the other end of the AC power supply into an electrical outlet. Always use caution when inserting the power supply to an electrical outlet. The Zen-O lite™ portable oxygen concentrator can also be powered using the DC power supply in a vehicle or comparable power source.

5. Connect the cannula to the oxygen outlet of the device. Follow the cannula manufacturer's instructions to place correctly on the patient.

6. To turn the device on, press the power key.

7. Select the flow rate prescribed by your clinician and breathe normally. The flow rate can be adjusted by pressing the + and - keys on the device.

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